



Quality Without Compromise.

Anzen™ Safety Scalpel has a reusable stainless steel handle similar in weight and balance to traditional metal handle scalpels, providing surgeons and techs with necessary safety features without losing the familiarity, control and ease-of-use of their current scalpels.

- Weighted and balanced to provide instant familiarity, control and comfort
- Intuitive design provides effortless blade exposure
- Slim cartridge design does not interfere with view of incision site
- Single-use protected blade cartridge makes sharps injury prevention easy and convenient
- Protected blade stays retracted when attaching and detaching cartridges

Simple to Use. Simple to Implement.

Anzen decreases workplace injuries by creating safer operating rooms with fewer sharps incidents from scalpels.


Anzen™ Safety Scalpel makes compliance with the Needlestick Safety Act effortless. By minimizing unnecessary exposure to the scalpel blade, you can ensure fewer preventable workplace injuries occur in the operating room and spend less time on the administrative pain following a sharps incident.

FDA-listed CE-Marked
Global patents issued and pending

Product Specifications


Blade Cartridges		
Product	Blade Size	Button Color
AZ10	#10	Blue
AZ11	#11	Green
AZ15	#15	Yellow

Reusable Handles		
Product	Handle Size	Material
AZH3	#3	Stainless Steel
AZH3L	#3L	Stainless Steel

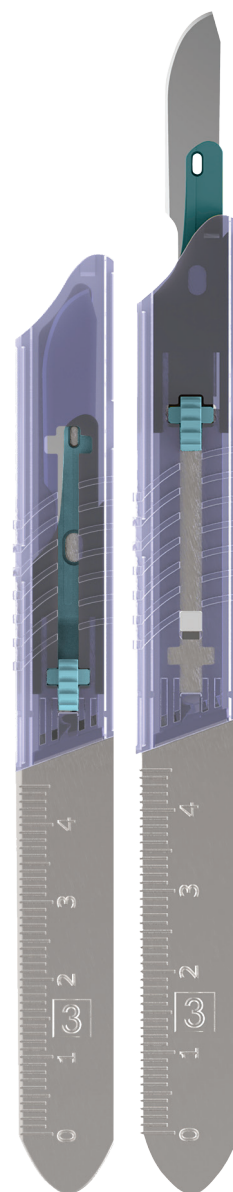


**ANZEN™ #10:
RETRACTED &
EXPOSED**

① Attaching Blade Cartridge

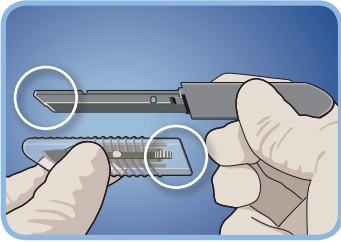

Align Handle and Cartridge orientations by matching end angles.

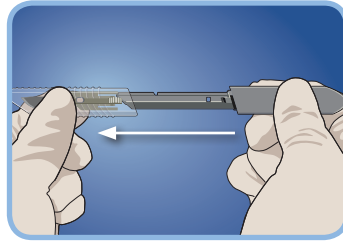


Insert Handle into Cartridge.

② Extending Blade




Depress Button on Cartridge.

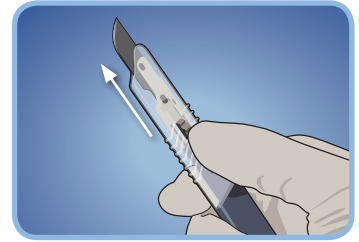

Slide Button forward until it locks.

③ Retracting Blade

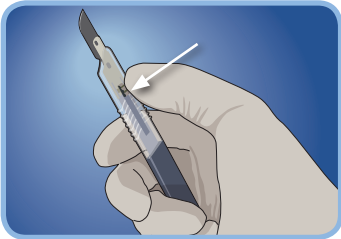


Depress Button on Cartridge.




Slide Button back until it locks.

④ Detaching Blade Cartridge

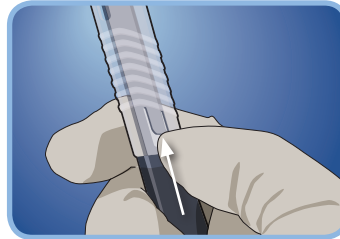


Lift Release Tab (Dorsal View).

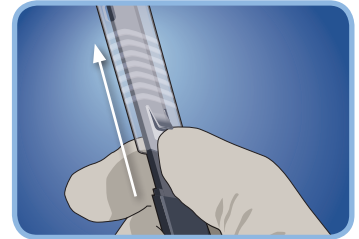


Slide cartridge forward to eject.

Ventral View (Button Side)

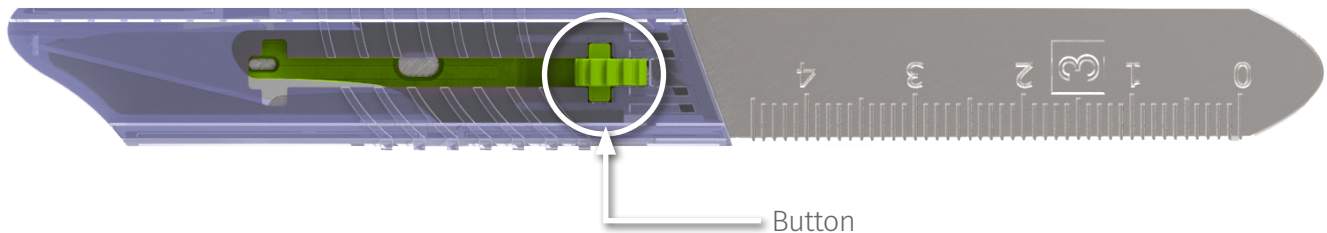

Dorsal View (Release Tab Side)

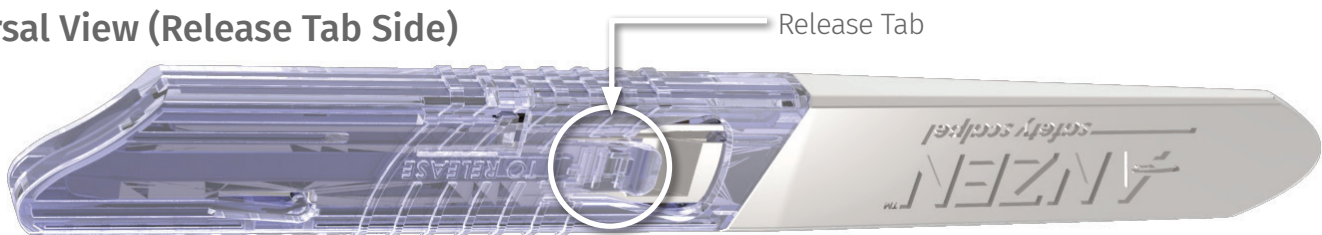

Visit anzenscalpel.com or contact us:

3883 Rogers Bridge Road NW
Suite 501
Duluth, Georgia 30097, USA

+1-404-855-3655 | tel
+1-770-559-4382 | fax
sales@medipurpose.com