



**1 Attaching Blade Cartridge**

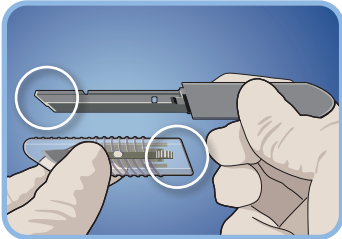


Align Handle and Cartridge orientations by matching end angles.

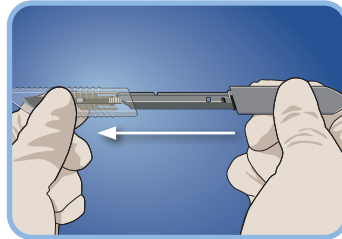


Insert Handle into Cartridge.

**2 Extending Blade**

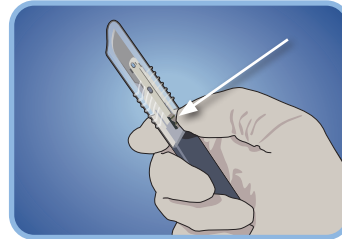


Depress Button on Cartridge.

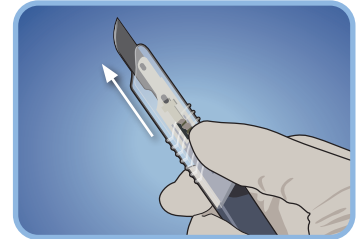

Slide Button forward until it locks.

**3 Retracting Blade**

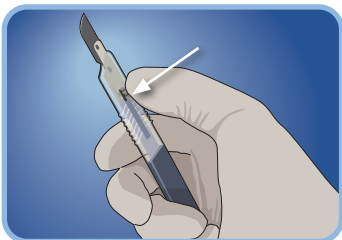


Depress Button on Cartridge.




Slide Button back until it locks.

**4 Detaching Blade Cartridge**

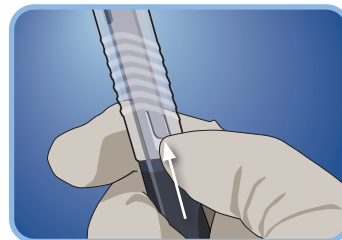


Lift Release Tab (Dorsal View).

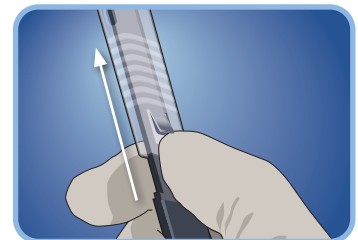

Slide cartridge forward to eject.


Cartridge is single-use only. Reuse may result in patient cross-contamination. **After use, discard into a sharps container.**

**Ventral View (Button Side)**

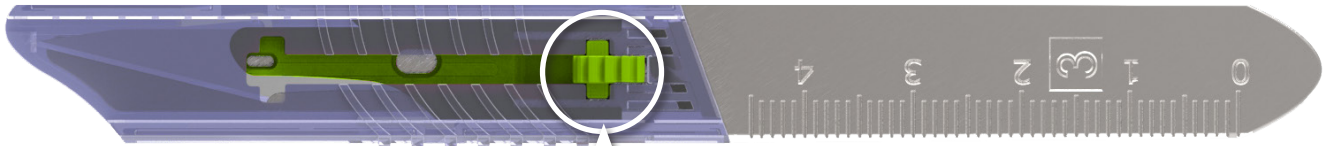

Button

**Dorsal View (Release Tab Side)**

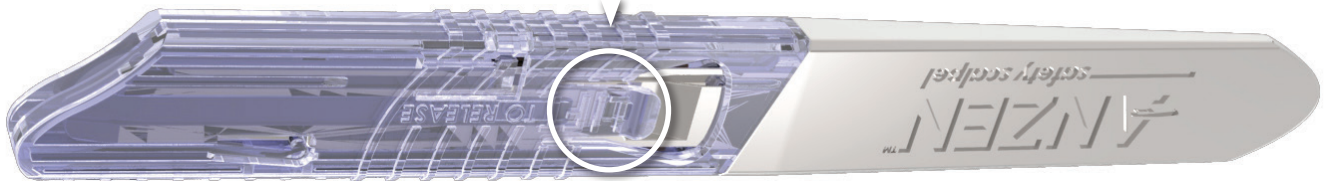

Release Tab

**1 Warning:** For single patient use only. Do not use if the package is broken; Do not store in high temperature, high humidity environment or with corrosive gas. Do not use if product is damp or expired.

2 The Anzen Safety Scalpel is designed to be used by medical professionals.

3 In the event of a serious incident, contact MediPurpose at [sales@medipurpose.com](mailto:sales@medipurpose.com) and/or the competent authority of the Member State in which the incident occurred.

Visit [anzenscalpel.com](http://anzenscalpel.com) or contact us:



MEDIPURPOSE PTE. LTD.  
10 Anson Road, #12-08  
International Plaza, Singapore 079903



Advena Ltd.  
Tower Business Centre, 2nd Flr.,  
Tower Street, Swatar, BKR 4013 Malta



3883 Rogers Bridge Road NW, Suite 501, Duluth, Georgia 30097, USA  
tel +1-404-855-3655 | fax +1-770-559-4382 | [sales@medipurpose.com](mailto:sales@medipurpose.com)

©2022 MediPurpose Pte Ltd

A product of **MediPurpose**<sup>™</sup>  
[www.medipurpose.com](http://www.medipurpose.com)