



**1 Attaching Blade Cartridge**

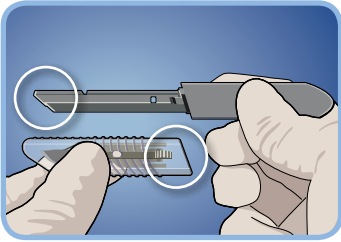


Align Handle and Cartridge orientations by matching end angles.

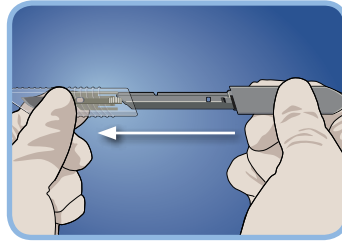


Insert Handle into Cartridge.

**2 Extending Blade**

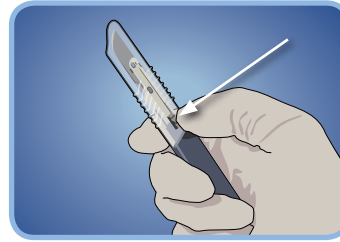


Depress Button on Cartridge.

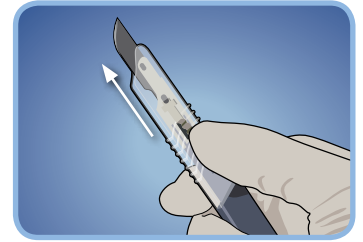

Slide Button forward until it locks.

**3 Retracting Blade**

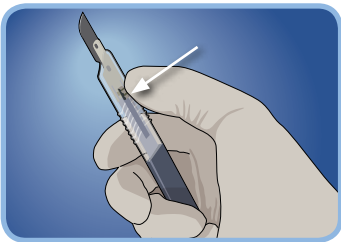


Depress Button on Cartridge.




Slide Button back until it locks.

**4 Detaching Blade Cartridge**

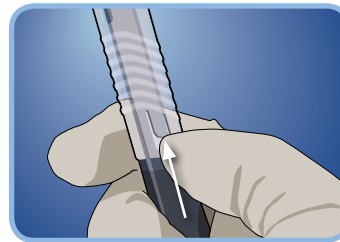


Lift Release Tab (Dorsal View).

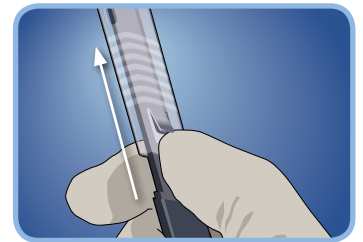


Slide cartridge forward to eject.

**Cartridge is single-use only. Reuse may result in patient cross-contamination.**

**Ventral View (Button Side)**

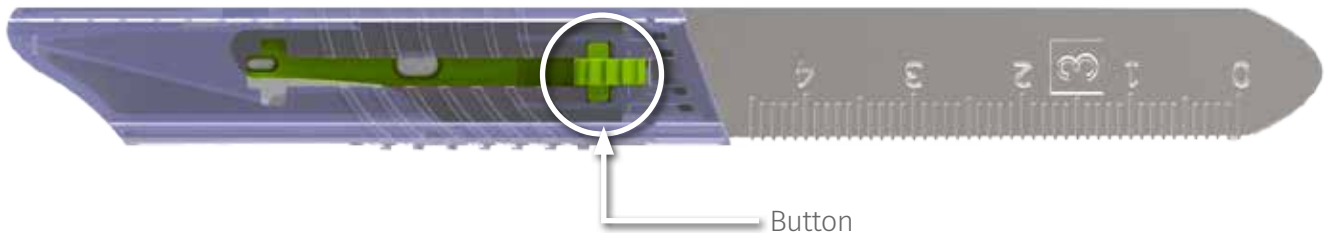

**Dorsal View (Release Tab Side)**

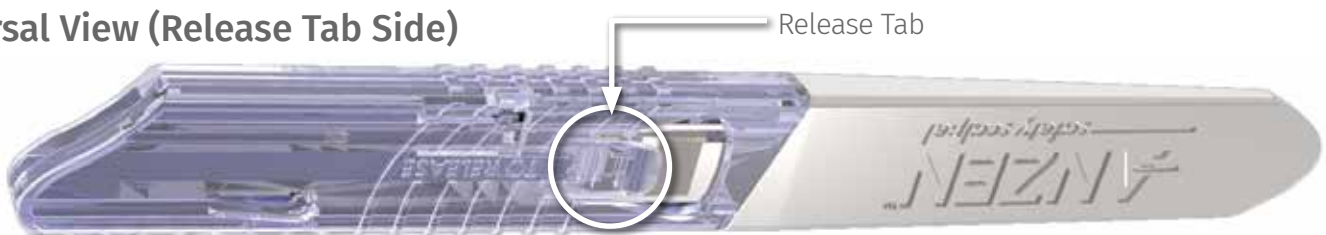

Visit [anzenscalpel.com](http://anzenscalpel.com) or contact us:

3883 Rogers Bridge Road NW  
Suite 501  
Duluth, Georgia 30097, USA

+1-404-855-3655 | tel  
+1-770-559-4382 | fax  
sales@medipurpose.com