



## 1 Attaching Blade Cartridge

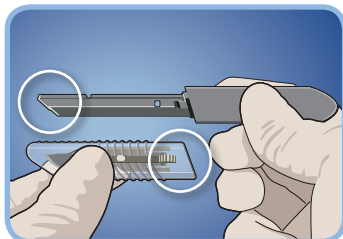

Align Handle and Cartridge orientations by matching end angles.

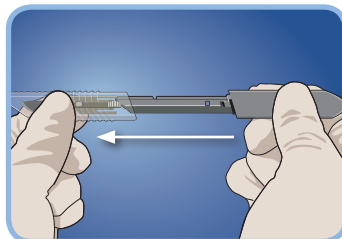


Insert Handle into Cartridge.

## 2 Extending Blade



Depress Button on Cartridge.

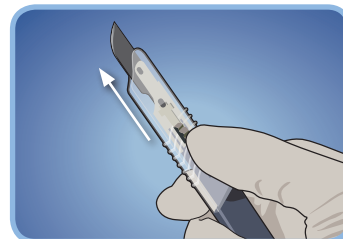


Slide Button forward until it locks.

## 3 Retracting Blade




Depress Button on Cartridge.

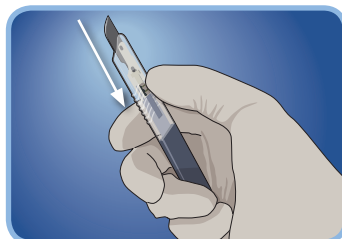


Slide Button back until it locks.

## 4 Detaching Blade Cartridge

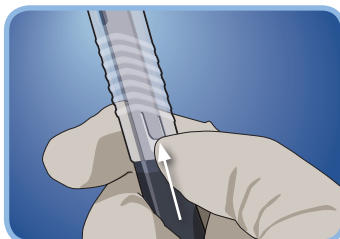


Lift Release Tab (Dorsal View).

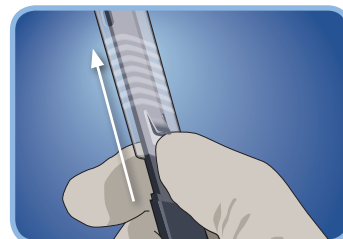

Slide cartridge forward to eject.


Cartridge is single-use only. Reuse may result in patient cross-contamination. **After use, discard into a sharps container.**

## Ventral View (Button Side)

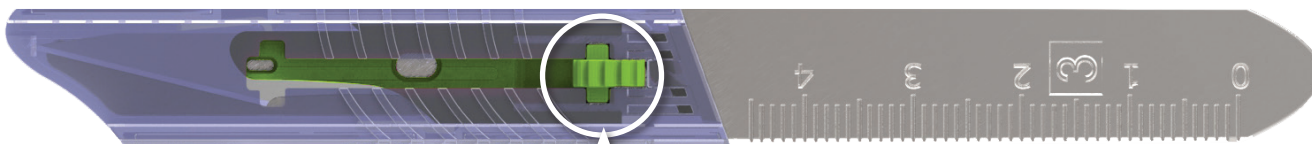


Button


## Dorsal View (Release Tab Side)

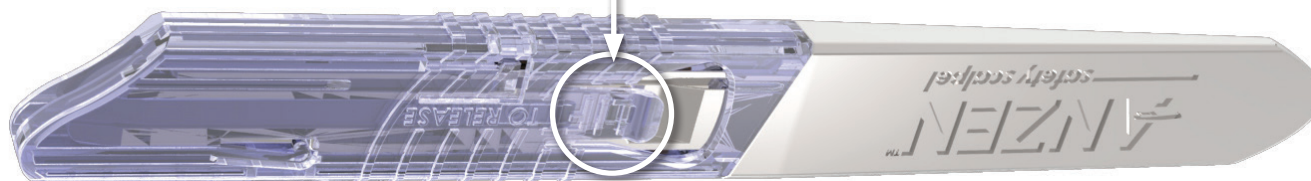

Release Tab

Visit [anzenscalpel.com](http://anzenscalpel.com) or contact us:

 MEDIPURPOSE PTE. LTD.  
10 Anson Road, #12-08  
International Plaza, Singapore 079903

 OBELIS S.A.  
Bd. Général Wahis, 53  
1030 Brussels, Belgium

 0197

3883 Rogers Bridge Road NW, Suite 501, Duluth, Georgia 30097, USA  
tel +1-404-855-3655 | fax +1-770-559-4382 | [sales@medipurpose.com](mailto:sales@medipurpose.com)

©2016 MediPurpose Pte Ltd

A product of **MediPurpose™**  
[www.medipurpose.com](http://www.medipurpose.com)